

Instructions for urine drug specimen collection

Urine drug test


Steps

1



Wash hands and put on any personal protective equipment.

2

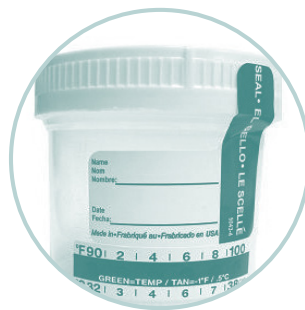

Place the following information on the collection container label:

Patient's name

Date of birth

Specimen collection date and time

Initials of the specimen collector

3

Provide the patient with a clean, unused labeled urine specimen collection container and instruct the patient to fill the container at least half full (a minimum of 10 mL's).

4

Accept the specimen from the patient. Urine temperature should be measured within four (4) minutes of collection and should read between 90-100°F.

5



Refer to page 28 for packaging and shipping instructions.