

Instruction for saliva sample collection

Acutis Reveal™ COVID-19 test


Materials



Collection notes

- Patient must not eat, drink, smoke or chew gum for 30 minutes prior to providing a sample
- Each device requires the following amount of saliva (to the fill line) without bubbles:
 - Spectrum = 2mL
 - Genotek = 1 mL
- Collection must not exceed the indicated volume
- Specimen must be stored at 15-30°C

Option 1: Spectrum device instructions

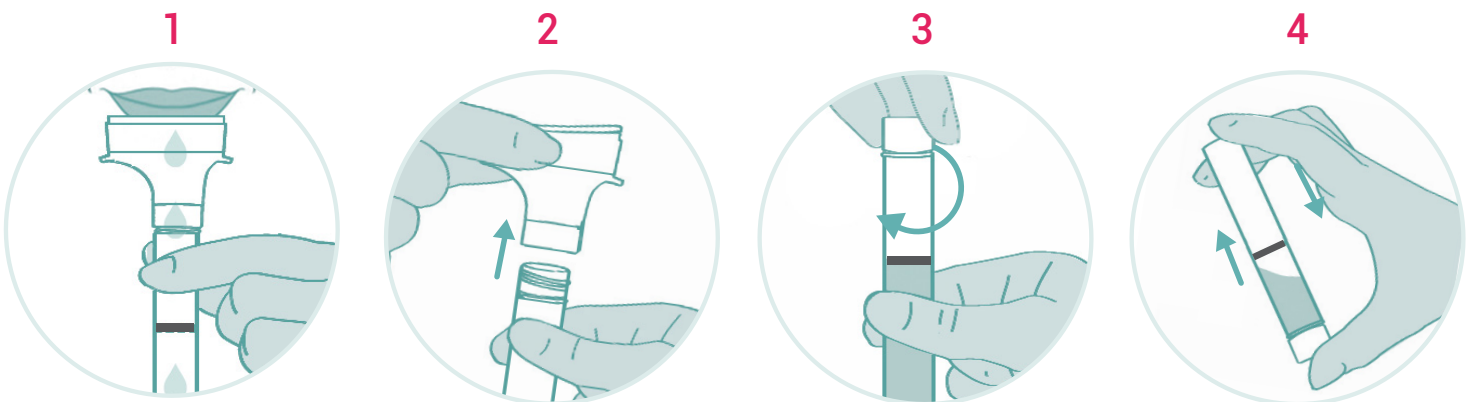

Fill the collection device with saliva to the black wavy line.


Remove the funnel and replace with the fluid filled cap.

Firmly screw the cap onto the collection device to release solution.

Shake tube for at least 5 seconds to mix the saliva and solution thoroughly.

Acūtis Reveal™ COVID-19

Option 2: Genotek device instructions

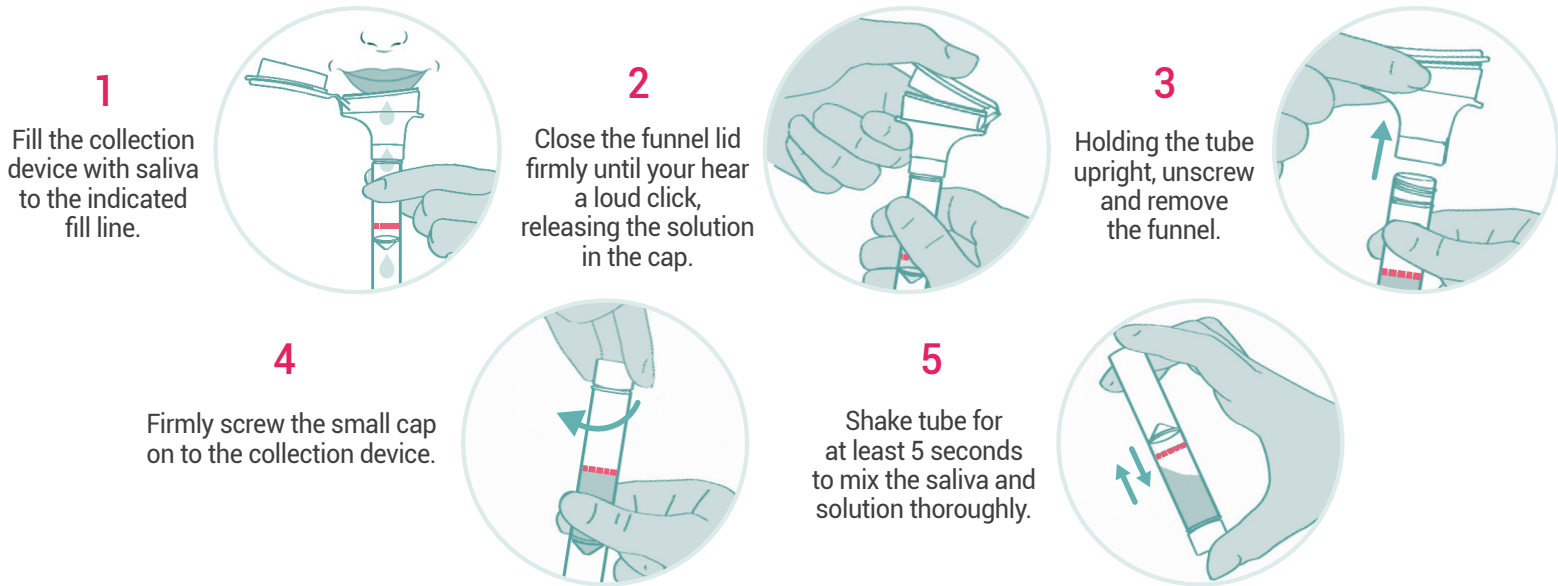

For both options



Mailing instructions

- Confirm patient name, DOB and collection date are legible on the vial
- Place tube into the sealed compartment of the specimen bag
- Be sure to include in the specimen bag the following:
 1. Test requisition form
 2. Patient demographics
 3. Copy of front and back of insurance card
 4. Medical records indicating medical necessity and prescribed medication(s)
- Place the completed specimen bag(s) in the provided protective box
- Place protective box containing specimen bag(s) into the FedEx / UPS Lab Pak and seal the Lab Pak
- Affix prepaid return label on the Lab Pak, and schedule a pickup
- Send to Acutis Diagnostics, 400 Karin Lane, Hicksville, NY 11801