


Instruction for sample collection

Acutis Reveal™ COVID-19 test


Due to national supply shortages related to COVID-19, nasopharyngeal swabs may be unavailable. Some swabs, typically indicated for other uses, have been tested and approved by the CDC for use in COVID-19 testing.

Clinician should determine method of collection based on swab type and the patient's nasopharyngeal anatomy.

Option 1: Determine if the singular nasopharyngeal swab can be used to collect a sample from the nasopharynx. If yes, proceed as follows:

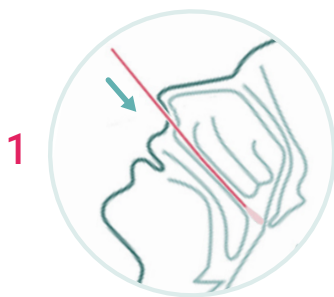


1 Insert swab into one nostril straight back, NOT upward, and horizontally to the nasopharynx until resistance is met.

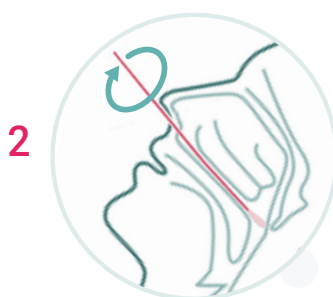

2 Rotate the swab up to five times and hold in place for 5 to 10 seconds to collect sample materials.




3 Insert the swab into the viral transport medium.

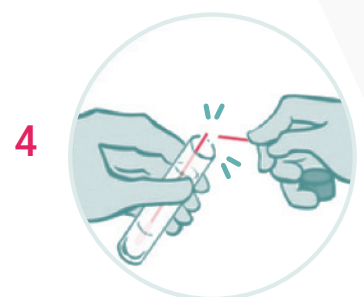

4 Break handles of swab at the breakpoint line.

Option 2: If the singular nasopharyngeal swab cannot be used to collect a sample from the nasopharynx, obtain a nasal as follows:


Note: this option can be self-administered by the patient.

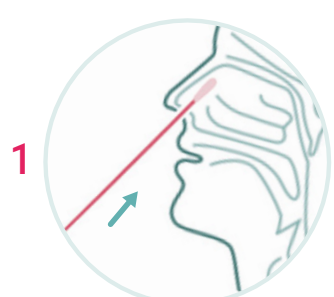


1 While gently rotating the swab, insert swab less than one inch into nostril.



2 Rotate the swab several times against nasal wall and repeat in other nostril using the same swab.

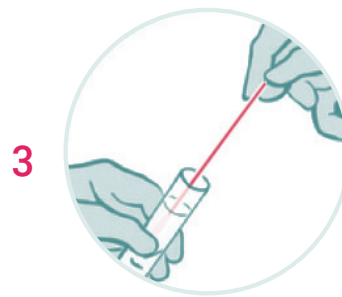

3 Insert the swab into the sterile transport media.

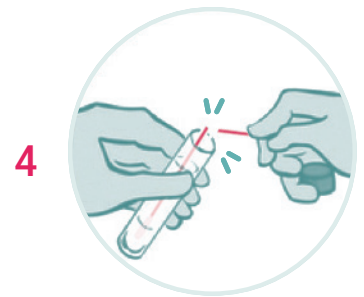

4 Break handles of swab at the breakpoint line.

Acūtis Reveal™ COVID-19

For both options

A




Replace the cap and tighten to secure.

B



Label vial with patient information and send to lab for testing.

C

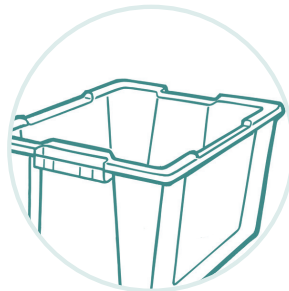

Dispose of PPE in appropriate waste receptacle.

D



Wash hands thoroughly.

Mailing instructions

- Confirm patient name, DOB and collection date are legible on the vial
- Place tube into the sealed compartment of the specimen bag
- Be sure to include in the specimen bag the following:
 1. Test requisition form
 2. Patient demographics
 3. Copy of front and back of insurance card
 4. Medical records indicating medical necessity and prescribed medication(s)
- Place the completed specimen bag(s) in the provided protective box
- Place protective box containing specimen bag(s) into the FedEx / UPS Lab Pak and seal the Lab Pak
- Affix prepaid return label on the Lab Pak, and schedule a pickup
- Send to Acutis Diagnostics, 400 Karin Lane, Hicksville, NY 11801