Acutis

Acutis Reveal[™] GIT gastrointestinal pathogen test

The simple truth: clinicians require faster and more accurate test results for gastrointestinal infections. They also want actionable data that prevents misdiagnoses and subsequent inappropriate use of antibiotics.

The good news: these higher quality results are achievable with Acutis Reveal[™], an FDA-approved process made available by a company known for delivering on its promises. Acutis Reveal[™] GIT can detect 14 major gastrointestinal pathogens. This one step, non-invasive test screens for infectious bacteria, viruses, and parasites as guickly as the same day the sample reaches our lab.

	Acutis Reveal [™] GIT	Stool Culture	Ova & Parasite
Sensitivity	96% (92-100%)	54% (48-60%)	72%
Specificity	98% (96-100%)	100%	100%
Additional confirmation requirements	No	No	Yes
Tests for Viral, Bacterial & Parasitic	Yes	Bacterial or Viral only	Parasitic only
Number of pathogens tested	14	1	1
Turnaround time	Within 24 hours of sample receipt	2-3 days	up to 3 days

The FDA-approved test detects 14 major gastrointestinal pathogens:

Simultaneous detection nd identification of >90% of the	Delivery of 95% of results the	Prevention of misdiagnoses and	Reduced unnecessated testing leads to decrease	
		:-	`- }-	
Parasitic targets	Cryptosporidium Entamoebe histolytica Giardia lamblia			
Bacterial targets	Campylobacter Clostridium difficile toxins A & B Enterotoxigenic E. coli (ETEC) LT/ST Escherichia coli 0157	Salmonella Shiga-like Toxin produ Shigella Vibrio cholera	Shiga-like Toxin producing <i>E. coli</i> (STEC) stx1/stx2 <i>Shigella</i>	
Viral targets	Adenovirus 40/41 Norovirus GI/GII Rotavirus A			

and bacteria, viral and parasitic pathogens responsible for infectious diarrhea

same day we receive the sample

subsequent inappropriate use of antibiotics

sarv eased downstream procedures (X-rays, CT scans, etc.)



Why choose Acutis Reveal[™]?

Acutis Reveal[™] gastrointestinal pathogen test can identify pathogens faster and with greater accuracy than any traditional methods or procedures. It detects and identifies >90% of infectious agents, providing the certainty clinicians require to provide the most efficient, efficacious, and economical patient care.

Gastroenteritis is a perpetual and growing problem. It too often frustrates the attending physician, always punishes the patient, and frequently puts others at risk. Acutis Reveal[™] offers a new, proven, state-of-the-art technique to diagnose gastroenteritis and the specific viral, bacterial or parasitic pathogens.

Gastroenteritis is the second most common disease after respiratory ailments, and a debilitating ailment that affects millions of people a year. It is also a leading cause of death of infants, the elderly and those with compromised immune systems, which is why diarrhea should not be allowed to run its course.*

Acutis Reveal[™] quickly determines whether or not cases of diarrhea are infectious. It determines the specific bacteria, viral and parasitic pathogens so that medical providers can then take a more specific course of action, for example if the bacterial toxin is identified as *E. coli* (STEC) or Shiga-like Toxin, the use of antibiotics may be appropriate. Or, if it is a case of *Clostridium difficile* toxins A and B, swift action may be called for due to its contagious nature. For certain other patients, such as those with IBD, the ability to rule out infections is especially useful.

In recent times, these particular pathogens have become notorious as they become better known to the general public. Our test goes some way to calm your patients' anxiety.

Molecular detection of infectious diseases eases and expedites the process

Acutis Reveal[™] correctly identifies sources of illness so that physicians and others won't be distracted by common symptoms such as abdominal pain, gas, constipation, or diarrhea.

This test also provides the necessary etiological distinctions, which lead to the most appropriate treatment. And, as these illnesses are often contagious or require immediate care, the faster results are returned, the quicker their threats can be contained. Please note, each test should only be ordered when deemed to be reasonable and medically necessary.



acutis.com

400 Karin Lane, Hicksville, NY 11801 68 Culver Road, Suite 150B, Monmouth Junction, NJ 08852 Fax 631-532-1680