


Instruction for nasopharyngeal in universal transport media

Acutis Reveal™ respiratory infection test (RIT)

Materials

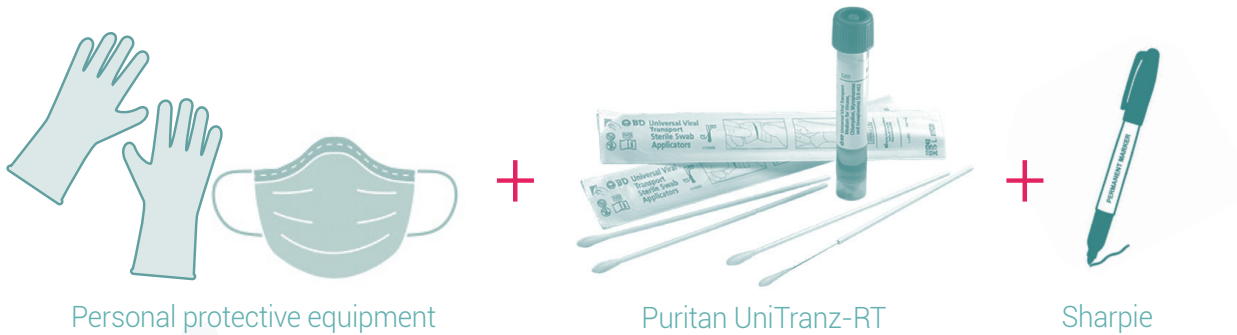

Steps

1



Wash hands and put on
any personal protective equipment

2



Peel open the Puritan UniTranz-RT
and remove the sterile flock swab from its package


Acūtis Reveal™ RIT

3



Tilt patients head back slightly to straighten nasal passage.

4

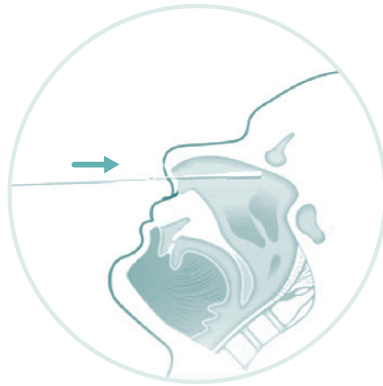

Insert swab into one nostril straight back, NOT upward, and horizontally to the nasopharynx until resistance is met.

5



Rotate the swab up to five times and hold in place for 5 to 10 seconds to collect sample materials.

6



Insert the swab into the viral transport medium and break handle at the breakpoint line.



7

Replace the cap and tighten to secure. Label vial with patient information and deliver samples to lab for testing.

Mailing instructions

- Confirm a legible label from the top of the Acūtis Reveal™ requisition is affixed to the specimen tube
- Place tube into the sealed compartment of the specimen bag
- Be sure to include in the specimen bag the following:
 - 1. Test requisition form
 - 2. Patient demographics
 - 3. Copy of front and back of insurance card
 - 4. Medical records indicating medical necessity and prescribed medication(s)
- Place the completed specimen bag(s) in the provided protective box
- Place protective box containing specimen bag(s) into the FedEx / UPS Lab Pak and seal the Lab Pak
- Affix prepaid return label on the Lab Pak, and schedule a pickup
- Send to Acūtis Diagnostics, 400 Karin Lane, Hicksville, NY 11801